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## seventy hyal 2000

### Solution of sodium hyaluronate for intradermal injection - Prefilled syringe

**Product description:** SEVENTY HYAL 2000 is an implantable medical device made of a sterile, pyrogen-free, viscoelastic solution of Sodium Hyaluronate, obtained by bacterial fermentation starting from a high molecular weight fraction (>2000 kDa). The Hyaluronic Acid, a polysaccharide of the glucosamine glycans family, is one of the basic components of the dermal extracellular matrix which gives the tissues their characteristic mechanical and elastic features.

The endogenous Hyaluronic Acid decreases with increasing age, contributing to the epidermal and dermal structures alteration, thus accelerating the skin aging and wrinkling.

SEVENTY HYAL 2000 delays the aging process by Hyaluronic Acid supplementation, increasing the skin hydration and creating the ideal conditions for the dermal tissue restoring.

**Composition:** Main ingredient: High Molecular Weight Sodium Hyaluronate 1.5%, obtained by bacterial fermentation without any chemical modification. Other ingredients: Sodium Chloride, Disodium Phosphate 2H<sub>2</sub>O, Monosodium Phosphate 2H<sub>2</sub>O, Water for Injection.

**Indications:** SEVENTY HYAL 2000 is intended for the treatment of cutaneous blemishes, particularly in the following areas:

- nasolabial folds
- peribuccal wrinkles
- glabellar wrinkles
- forehead wrinkles
- face scars including acne scars
- face areas needing a treatment (cheeks, chin, cheekbones, lips) able to induce a temporary volume increase by soft tissues swelling.

High molecular weight Hyaluronic Acid, by its hygroscopic and viscoelastic properties, improves tissues hydration and promotes their remodelling with a consequent corrective action on both tonicity and scars. The direct supplementation inside the dermal tissue allows the right amount of Hyaluronic Acid needed to prevent and repair skin aging processes to be fully available.

**Dosage and method of administration:** Inject SEVENTY HYAL 2000, using a suitable sterile needle (30 G) directly inside the dermal tissue. The injection should be exclusively done by a medical practitioner. The injection site should be on the intact skin.

All the precautions relating to asepsis and good injection practice should be observed.

The infiltration should be done inside the medium/deep dermal tissue; anyway the procedure is at the physician's discretion and depends on the correction and the adopted technique. The prefilled syringe of SEVENTY HYAL 2000 contains 2 ml product, therefore it is advisable to substitute the needle at half-infiltration.

At the end of the injection a delicate massage of the treated area can help the product being uniformly absorbed.

It is advisable that the treatment is customised based on each patient's skin and expectations.

**Contraindications:** Do never administer to patients having a known hypersensitivity to any of the product ingredient, in case of skin infectious disease or pathology at the injection site, in case of blood coagulation diseases or during a therapy with anticoagulants.

**Special warnings:** The prefilled syringe content is sterile. Any prefilled syringe is packed inside a sealed blister. The external part of the syringe is not sterile but its microbial count is controlled.

The intra-dermal infiltration should be exclusively done by a medical practitioner.

Though preclinical trials on laboratory animals indicate that the product has no potential toxicity on reproduction and foetal development, SEVENTY HYAL 2000 has actually never been tested on pregnant women.

Do not use in case of damaged package.

Do not use after the expiry date reported on the package. The expiry date refers to the product stored in its original package at a temperature not higher than 25°C and protected from light.

SEVENTY HYAL 2000 is intended for a single dose administration; dispose of it after its application in accordance with local requirements. Keep out from the reach and sight of children.

**Side effects:** Local soreness, itching, swelling or redness at the injection site may occasionally be present. Such symptoms are usually mild and transient.

For a quicker recovery it is advisable to apply ice at the injection site.

**Interactions:** Do not concomitantly use disinfectants containing quaternary ammonium salts because Hyaluronic Acid can precipitate in their presence.

SEVENTY HYAL 2000 can be concomitantly used with local anaesthetics containing lidocaine.

**Storage conditions:** Store at a temperature not higher than 25°C and protect from light

**Packaging:** box containing 3 prefilled syringes. Each syringe is singularly sealed in a blister and contains 30mg/2ml of Sodium Hyaluronate sterilised by moist heat.

 **Manufacturer:** SAVIO INDUSTRIAL S.r.l. Via del Mare, 36 – 00071 Pomezia (RM)

 **0373**

Distributed by: **Seventy BG srl** – Via Albicri 3 – 20122 Milano – Tel. +39 02 3657 2070 – Fax +39 02 3657 2074 – www.seventybg.it - info@seventybg.it

Seventy Hyal 2000 - Date of last revision October 2019

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Manufacturer: SAVIO INDUSTRIAL S.r.l. Via del Mare, 36 – 00071 Pomezia (RM)